


# LazaMatic/UltraMatic

## Instructions for use

### Lazamatic (Blue)


1

To load, depress spring clip and insert female tag.

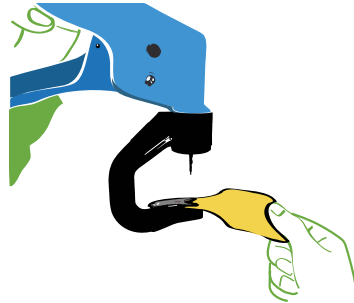


2

Slip male button completely onto application pin. It is recommended that adult cattle are restrained in a head bail for tagging.

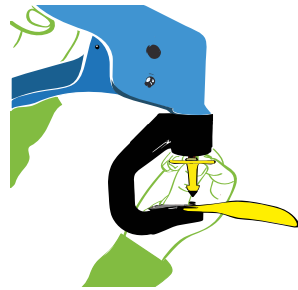


3

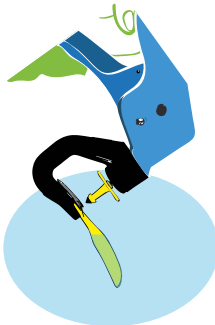


Dip jaws of applicator holding tag into antiseptic or disinfectant solution. NB: Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.

4

With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear.

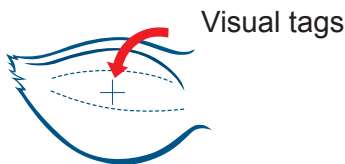


1

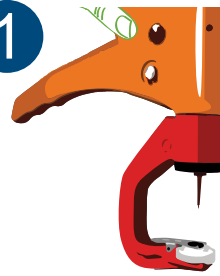


The female NLIS Tag has a lump on its printed face and an arrow on the underside. To load, depress spring clip and insert female tag, holding the lump between thumb and forefinger. Position the tag so the lump faces out of the open end of the applicator. Failure to do so may result in damage to the electronic tag.

2

Slip male button completely onto application pin. It is recommended that adult cattle are restrained in a head bail for tagging.

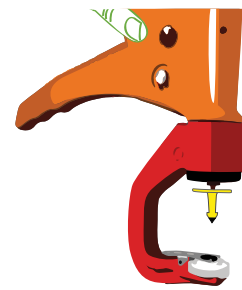


3

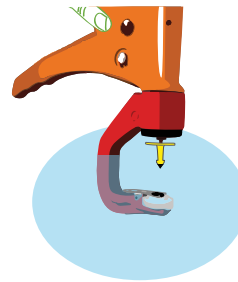

Dip jaws of applicator holding tag into antiseptic or disinfectant solution. NB: Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.

4

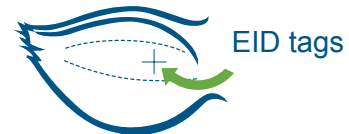

With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear. Electronic tags should be attached in the centre between the two cartilage bones of the inside third of the ear (as shown above).



It is recommended the electronic tag is applied to the RIGHT (the animal's right side) ear as per the illustration above.

### UltraMatic (Orange)