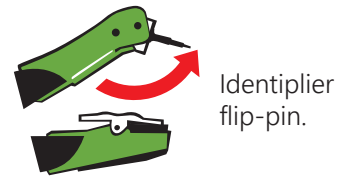


Identiplier/Universal

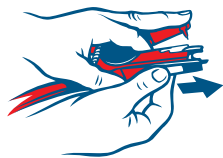
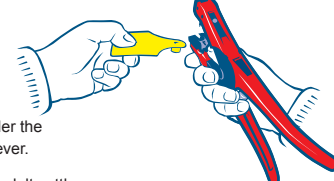
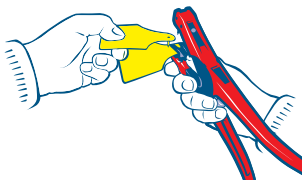
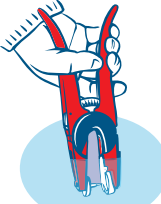
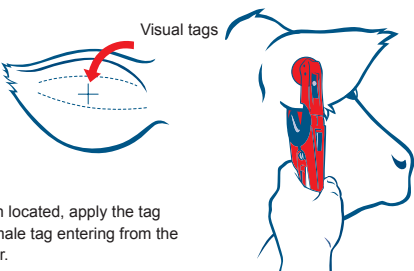
Instructions for use

When do I use the green vs the red applicator?


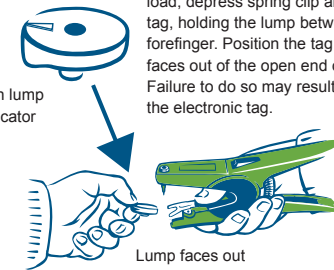
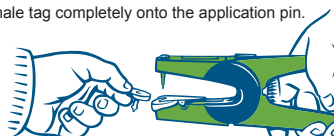

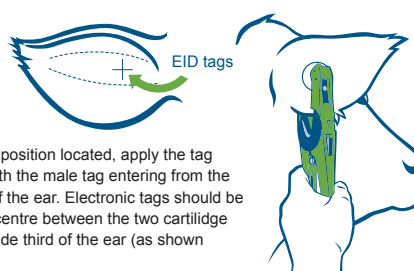
Either applicator can be used for Electronic cattle and sheep tags and visual tags. The difference between the applicators is that the Identiplier (green) has a swivel-action pin to prevent tearing should the animal pull away. The Universal applicator (red) has a static pin.



Universal (Red)

- 1  Ensure that the black insert is removed from the jaws of the Universal applicator.
- 2  Place the visual tag under the clip by depressing the lever.
It is recommended that adult cattle are restrained in a head bail for tagging.
- 3  Slip male tag completely onto the application pin.
- 4  Dip jaws of applicator holding tag into antiseptic or disinfectant solution.
NB: Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.
- 5  Visual tags
With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear.

Identiplier (Green)

- 1  Ensure that the black insert is removed from the jaws of the Identiplier applicator.
- 2  Female tag with lump (microchip) indicator arrow
The female NLIS has a lump on its printed face and an arrow on the underside. To load, depress spring clip and insert female tag, holding the lump between thumb and forefinger. Position the tag so the lump faces out of the open end of the applicator. Failure to do so may result in damage to the electronic tag.
Lump faces out from applicator
- 3  Slip male tag completely onto the application pin.
- 4  Dip jaws of applicator holding tag into antiseptic or disinfectant solution.
NB: Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.
- 5  EID tags
With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear. Electronic tags should be attached in the centre between the two cartilage bones of the inside third of the ear (as shown above).
It is recommended the electronic tag is applied to the RIGHT (the animal's right side) ear as per the illustration above.

