GPR+

GLOBAL POCKET READER™ PLUS

UNIVERSAL MICROCHIP READER FOR
ALL COMPANION ANIMAL

Quick Start Guide

MSD


Animal Health **Intelligence**

DESCRIPTION

GPR+ is a **universal portable reader** for RFID microchips with the purpose of identifying animals.

The device can read a **wide range of technologies**: FDX-B ISO, HDX ISO, FDX-A (FECAVA), Trovan and Avid encrypted. **GPR+** can also read and display the **Microchip Temperature™** when reading temperature sensing microchips*.

In addition to its microchip reading functions, the **GPR+** scanner can store up to **3000 IDs** in its internal memory, each associated with a time/date stamp, which can be transmitted to a personal computer via a USB cable or wirelessly using Bluetooth®.

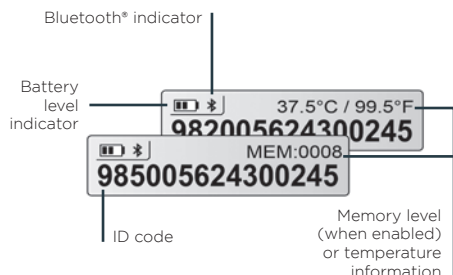The **GPR+** is supplied with the Microchip Management Software which enables user to download the reader's memory, change the scanner settings or update the **GPR+** firmware. It is available for download at www.destronfearing.com with the complete User Manual and Software User Guide.

1 Microchip Temperature™ range is from 33°C (91.4°F) to 43°C (109.4°F). If the temperature is below or above this range, it will display "TEMP BELOW RANGE" OR "TEMP ABOVE RANGE".

* for temperature sensing microchips sold by MSD Animal Health and its affiliates

GETTING STARTED

GPR+ comes supplied with six AAA **rechargeable NiMH batteries** which must be **fully charged prior to use**, as diagram below (step 1 through 3).

If preferred, six standard AAA alkaline batteries can also be used and should not be charged (follow only step 1).

Step 1 – Remove retention screw located on bottom back of **GPR+**. Place batteries according to polarity indicators. Replace lid and screw.

Step 2 – Remove USB plug and attach cable to **GPR+**.

Step 3 – Plug the adaptor into power outlet. The battery level icon will flash while charging and stop flashing when fully charged. A full charge will take approximately 3 hours.

DO NOT MIX RECHARGEABLE AND STANDARD BATTERIES.
DO NOT ATTEMPT TO CHARGE STANDARD ALKALINE BATTERIES.
CHARGE THE BATTERIES INDOORS ONLY AND BETWEEN 0°C AND 40°C.

To extend battery life, charge unit at least every three months, if reader is unused.

READING MICROCHIPS


When scanning for microchips **PRESS AND HOLD BUTTON DOWN** during procedure. The reader will display "SEARCHING".

Test reader by placing test chip provided on key chain. If microchip number is not detected, consult USER MANUAL. Do not attach key chain to reader.

If a microchip is detected, the number will appear on the display. If a microchip is not detected it will display "NO ID FOUND".

For Pets

Keep scanner touching animal and scan slowly following patterns below:

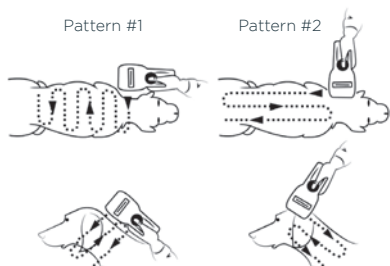For other species

Keep scanner touching animal over standard location for particular species.

Note 1 – Microchip placement may vary per animal.

MICROCHIP MANAGEMENT SOFTWARE


GPR+ scanner comes with Microchip Management Software only for Windows® operating system. Please refer to Microchip Management Software User Guide to install the tool on your PC.

Note 2 – **GPR+** is compatible with **iPhone®**, **iPad®** and **Android™**. Refer to the User Manual for more information.

Microchip Management Software offers the following capabilities:


- Change reader settings.
- Download reader memory and export animal ID's to Excel.
- Use reader as a virtual keyboard.
- Update reader firmware.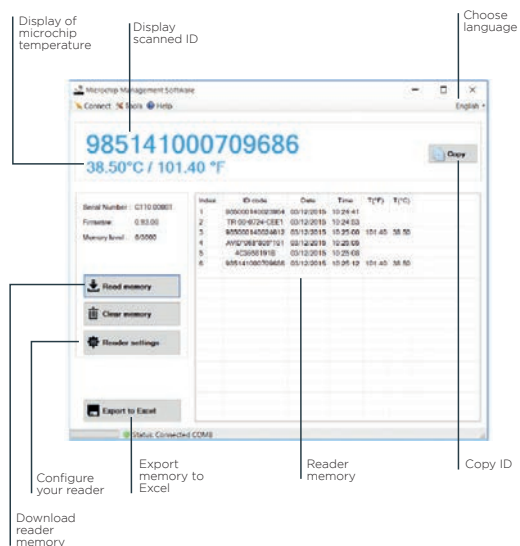

READER SETTINGS IN MICROCHIP MANAGEMENT SOFTWARE

GPR+ is delivered with Memory and Bluetooth® enabled.

These options can be disabled through the Microchip Management Software, as in diagram below:

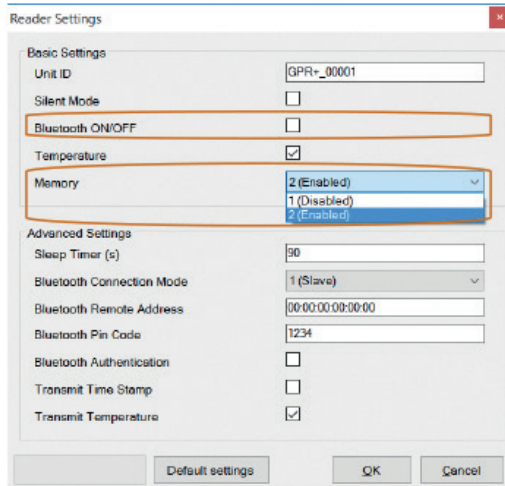TROUBLESHOOTING

- **Microchip not detected:**

Confirm that it is being operated distant from metal objects and computer equipment, including metal objects, such as collar with metal objects and stainless steel tables.

- **Memory cannot be downloaded:**

Make sure memory is enabled in reader settings (refer to Microchip Management Software User Guide for more information).

- **Bluetooth not visible:**

Check if Bluetooth icon appears on the display. If not, enable Bluetooth by using Microchip Management Software.

- **Bluetooth visible but not connectable:**

Make sure your reader is configured to be visible and connectable in Microchip Management Software (refer to Microchip Management Software User Guide for more information).

- **Incorrect date/time:**

Connect reader to Microchip Management Software. This will synchronize the date and time with the computer.

Note 3 - For other troubleshooting questions, consult user manual.

WARRANTY

The **Global Pocket Reader Plus** is warranted against defects in materials and workmanship, under normal use and service for one (1) year from the day of shipment.

This warranty will not apply if adjustment, repair or parts replacement is required because of accident, neglect, damage during transportation or causes other than ordinary use.

Manufacturer's sole responsibility under this warranty shall be at its option, to either repair or replace any product which fails during the warranty period. In no event shall Manufacturer be liable for any indirect or consequential damages or loss of profit.

LEGAL INFORMATION

Trademarks

Bluetooth® is a registered trademark of Bluetooth SIG, Inc.

Windows is a trademark or registered trademark of Microsoft Corporation in the United States and/or other countries.

All other trademarks are trademarks of their respective owners.

Apple - Legal Notice

iPhone, iPad are a trademark of Apple Inc., registered in the U.S. and other countries.

"Made for iPhone," and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPhone, or iPad, respectively, and has been certified by the developer to meet Apple performance standards.

Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards.

Please note that the use of this accessory with iPhone or iPad may affect wireless performance.

Snapshots are according to the latest version at the moment this document was printed.

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iPhone, iPad são uma marca comercial da Apple Inc., registada nos EUA e noutros países.